ISTRUZIONI D'USO SISTEMA IMPLANTARE I-RES SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY e SHAPEMINI

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PRODUCT CHARACTERISTICS

The SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant systems, by I-RES SAGL offers the dentist a wide choice of titanium implant configurations that differ in diameter, height and position of surgical positioning A) submerged/bone level, B) transmucoso/tissue level, and various prosthetic components for the different rehabilitation procedures.

INDICATIONS FOR USE

The SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant systems are indicated for surgical treatment in the upper or lower jaw for the partial or total replacement of teeth in patients who have lost part or all of their teeth. The implant to be used must be chosen by the medical personnel based on the medical history and on the subsequent surgical and prosthetic plan required for each individual patient. The one-part implants IMAXMUA, having the same geometric shape of IMAX dental implants, ensure an excellent retention of the prosthesis, thanks to the ability to accommodate the retained screw designed for MUA components with a pitch of 1/72 instead of 1,4 mm of the classic MUA. The implants are delivered in sterile packs and the operator must pay great attention when positioning it in the oral cavity, so that the implant does not come in contact

with elements that could alter the surface, hindering the healing processes, so all manoeuvres must be performed in an environment suitable for surgical activities. The SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant system has a series of dedicated surgical instruments for its implant lines, useful for the non-traumatic preparation of the site that is to receive the implant, and instruments designed for better treating peri-implantitis. Decontamination is performed with cold Argon plasma followed by packaging in a cleanroom, for the final sterilisation phase with gamma or beta rays. The pack containing the implant and the respective cover screw must be opened in a sterile environment in the phases of surgical implant therapy. I-RES SAGL implants are DISPOSABLE devices. Their reuse is not desirable from a medical, legal and ethical point of view. The use of not validated sterilization procedures can cause both the infection onset in the patient and impair the product performances. The failure of the implant, with the instructions implies, a different use as provided by the manufacturer and the quones who make the reuse must take this action on their own responsibility.

CONTRAINDICATIONS

Do not use SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant systems in patients and in any case where there is no suitable to guarantee the primary stability of the implant in the first phase of insertion, in patients with poor oral hygiene, smokers, with uncontrolled systemic pathologies and blood disorders. In addition to the normal contraindications for general surgery, the conditions described above may have a negative influence on the partial or total integration of the implant.

WARNINGS

To use the SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant systems by I-RES SAGL, the dentist must know the general surgical and prosthetic techniques and the specific techniques for SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI. Following the instructions, the dentist must use the specific training courses. An incorrect choice of implant and surgical technique can be prejudicial to the success of the operation, causing the loss of the implant and of the surrounding bone. No implant must be used that has been used previously, or that has come in contact with the organic tissues of the patient. The sterility of the implant is guaranteed by the sealed packaging and by correct storage in controlled dry environments; packages that are not intact or damaged are prejudicial to the use of the implant. For product traceability it is important to keep the batch number marked on the implant package and on the adhesive labels to be found in the same package. For the same reason it is recommended that the dentist keep as long as possible his patients' medical files, in which he has a record of their medical history, treatment plans, types of operations and prosthetic rehabilitations performed. The sterility of the implant can be useful for the patient's medical history. The use of non-original I-RES instruments is not advised, as is the failure to follow the indications for inserting the SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant systems and the reuse of prosthetic components. The instructions have been designed to obtain the best result: SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant systems must be inserted with a maximum torque of 50 Ncm, exceeding this force could be prejudicial to the precision of connection with the subsequent prosthetic part. The SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant systems include, in their range of implants, very small diameters made not only in pure titanium but also in titanium alloy (such as Ø 3,3 mm, Ø 3,7 mm in the connections internal and for Ø 2,7 mm for SHAPEMINI mini-implants), which must be used as implants only in the front of the mouth and not in diastoric areas where there is great masticatory stress. Especially the mini implants, with a Ø 2,7 mm, may be used only for the anchorage of the prosthesis. Furthermore, the implants with Ø 3,7 mm must not be inserted individually on premolars and molars, but at most should be only linked with bars to distribute the loading force. SHAPEMINI mini-implants may be used only in the front part of the mouth for single tooth replacement and not in the rest of the mouth. The implants with Ø 3,7 mm are higher, in this sites, they can only be used for dentures anchoring.

THE COMPANY I-RES SAGL DISCLAIMS HERSELF FOR ANY LIABILITY DUE TO THE NON OBSERVANCE OF THE INDICATIONS REPORTED IN THIS INSTRUCTION LEAFLET.

COLLATERAL EFFECTS

The known possible collateral effects are the partial or total failure of osseointegration, with consequent loss of the prosthetic function for which the implant system is intended, pain and transient paresthesia, fracture due to excessive load on the implant system, post or prosthesis.

PRE-OPERATIVE PLANNING

The careful study and assessment of patients who are candidates for implant prosthetic therapy is of fundamental importance. Physical, instrumental, and radiological examinations and the study of models allow the dentist to make the best diagnosis and consequent therapy. The preparation of the patient for surgical implant therapy and the preparation of the operating room must be given the same care and attention as general surgery; the preparation of the implant site using dedicated drills with controlled revolutions, cooled with saline solution, these are all indispensable conditions for implant therapy.

THE COMPANY I-RES SAGL DISCLAIMS HERSELF FOR ANY LIABILITY DUE TO THE NON OBSERVANCE OF THE INDICATIONS REPORTED IN THIS INSTRUCTION LEAFLET.

COLLATERAL EFFECTS

The known possible collateral effects are the partial or total failure of osseointegration, with consequent loss of the prosthetic function for which the implant system is intended, pain and transient paresthesia, fracture due to excessive load on the implant system, post or prosthesis.

PRE-OPERATIVE PLANNING

The careful study and assessment of patients who are candidates for implant prosthetic therapy is of fundamental importance. Physical, instrumental, and radiological examinations and the study of models allow the dentist to make the best diagnosis and consequent therapy. The preparation of the patient for surgical implant therapy and the preparation of the operating room must be given the same care and attention as general surgery; the preparation of the implant site using dedicated drills with controlled revolutions, cooled with saline solution, these are all indispensable conditions for implant therapy.

The SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant systems are produced in commercially pure grade 4 titanium - ASTM F67 - and in grade 5 titanium alloy (Ø 3,3 and Ø 3,7 in internal hexagon connections) – ASTM F136. SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implants are surface treated by sandblasting followed by means of sandblasting followed by double acid-etching. In the market are also available implants with different surface treatments depending on the intended use of the product, as well as implants with a final coating with hyaluronic acid for a better bone tissue healing. The only machined implants are suitable for the patient with periodontitis. The Hybrid implants (presenting a surface half machined and half superficially treated) are specified both for patients with periodontitis and for all those patients where greater care is necessary aimed at reducing or better treating peri-implantitis. Decontamination is performed with cold Argon plasma followed by packaging in a cleanroom, for the final sterilisation phase with gamma or beta rays. The pack containing the implant and the respective cover screw must be opened in a sterile environment in the phases of surgical implant therapy. I-RES SAGL implants are DISPOSABLE devices. Their reuse is not desirable from a medical, legal and ethical point of view. The use of not validated sterilization procedures can cause both the infection onset in the patient and impair the product performances. The failure of the implant, with the instructions implies, a different use as provided by the manufacturer and the quones who make the reuse must take this action on their own responsibility.

CONTRAINDICATIONS

Do not use SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant systems in patients and in any case where there is no suitable to guarantee the primary stability of the implant in the first phase of insertion, in patients with poor oral hygiene, smokers, with uncontrolled systemic pathologies and blood disorders. In addition to the normal contraindications for general surgery, the conditions described above may have a negative influence on the partial or total integration of the implant.

WARNINGS

To use the SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant systems by I-RES SAGL, the dentist must know the general surgical and prosthetic techniques and the specific techniques for SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI. Following the instructions, the dentist must use the specific training courses. An incorrect choice of implant and surgical technique can be prejudicial to the success of the operation, causing the loss of the implant and of the surrounding bone. No implant must be used that has been used previously, or that has come in contact with the organic tissues of the patient. The sterility of the implant is guaranteed by the sealed packaging and by correct storage in controlled dry environments; packages that are not intact or damaged are prejudicial to the use of the implant. For product traceability it is important to keep the batch number marked on the implant package and on the adhesive labels to be found in the same package. For the same reason it is recommended that the dentist keep as long as possible his patients' medical files, in which he has a record of their medical history, treatment plans, types of operations and prosthetic rehabilitations performed. The sterility of the implant can be useful for the patient's medical history. The use of non-original I-RES instruments is not advised, as is the failure to follow the indications for inserting the SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant systems and the reuse of prosthetic components. The instructions have been designed to obtain the best result: SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant systems must be inserted with a maximum torque of 50 Ncm, exceeding this force could be prejudicial to the precision of connection with the subsequent prosthetic part. The SHAPEI, IMAX, VOLUTION, IMAXMUA, HANDY AND SHAPEMINI implant systems include, in their range of implants, very small diameters made not only in pure titanium but also in titanium alloy (such as Ø 3,3 mm, Ø 3,7 mm in the connections internal and for Ø 2,7 mm for SHAPEMINI mini-implants), which must be used as implants only in the front of the mouth and not in diastoric areas where there is great masticatory stress. Especially the mini implants, with a Ø 2,7 mm, may be used only for the anchorage of the prosthesis. Furthermore, the implants with Ø 3,7 mm must not be inserted individually on premolars and molars, but at most should be only linked with bars to distribute the loading force. SHAPEMINI mini-implants may be used only in the front part of the mouth for single tooth replacement and not in the rest of the mouth. The implants with Ø 3,7 mm are higher, in this sites, they can only be used for dentures anchoring.

THE COMPANY I-RES SAGL DISCLAIMS HERSELF FOR ANY LIABILITY DUE TO THE NON OBSERVANCE OF THE INDICATIONS REPORTED IN THIS INSTRUCTION LEAFLET.

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